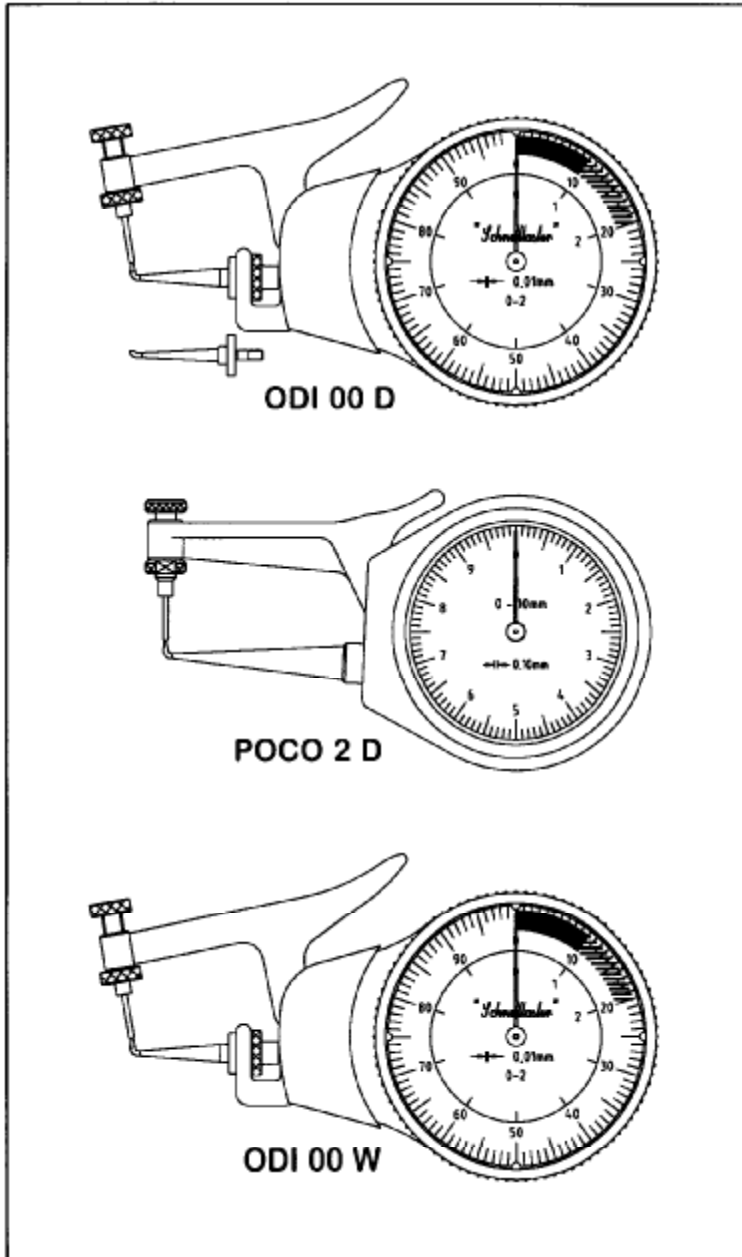
- D** Bedienungsanleitung
- GB** Operation instructions
- F** Mode d'emploi

ODI 00 D • POCO 2 D • ODI 00W

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*Subject to technical alterations.
If because of this instruction, rules in the
Product Liability Act are referred to in case of
damage these will herewith be explicitly
excluded.*

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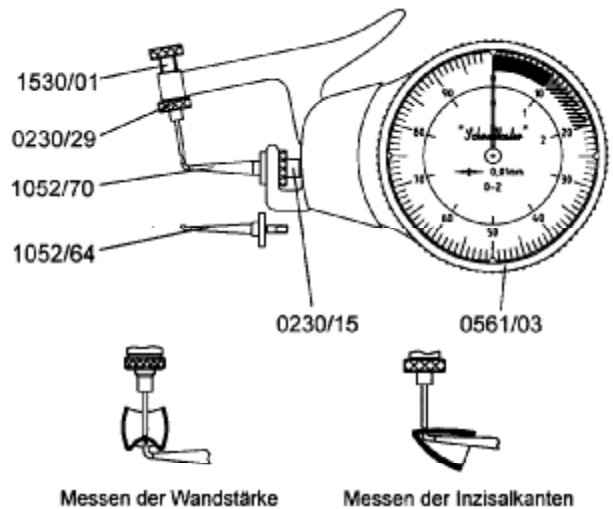
GB

The standard supply of the gauge ODI 00 D includes the lower caliper arm no. 1052/70 for the measurement of wall thicknesses and fissures. This arm is assembled and adjusted. Another caliper arm for the measurement of incisal margins, no. 1052/64, is also included, but not assembled.

Changing the caliper arm:

1. Loosen the knurled nut 0230/15 and remove the caliper arm.
2. Insert the other arm in the base and tighten the knurled nut. Please see to it that the pin at the base fits exactly into the groove of the caliper arm.
3. Adjust the instrument :
 - Loosen the knurled nut no. 0230/29 at the upper measuring contact.
 - Turn the upper measuring contact no. 1530/01 so that both pointers indicate approximately the zero position.
 - Tighten the knurled nut 0230/29.
 - Turn the scale until the big pointer indicates the zero position. The small pointer does not have to indicate exactly the zero position.

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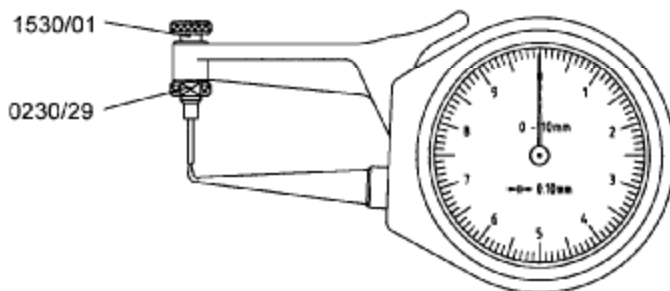
GB

The gauge POCO 2 D is used for the measurement of wall and base thicknesses of crowns and fissures.

Renew the measuring contact 1530/01:

1. Loosen the knurled nut no. 0230/29 and remove the measuring contact.
2. Insert the new contact and tighten the knurled nut.
3. Adjust the instrument:
 - Loosen the knurled nut no. 0230/29 at the upper measuring contact.
 - Turn the upper measuring contact no. 1530/01 so that the pointer indicates the zero position.
 - Tighten the knurled nut no. 0230/29.

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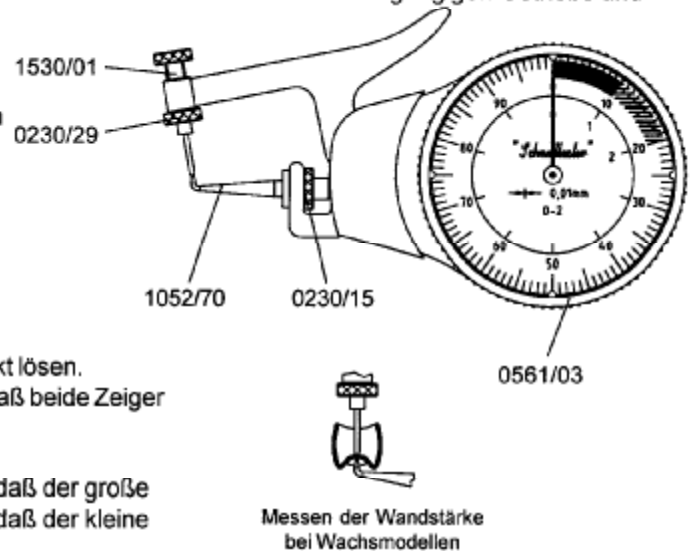
- GB** The gauge ODI 00 W is used for the measurement of wax thicknesses. It has a very smooth mechanism and a small measuring pressure.

Renew the caliper arm 1052/70:

- Loosen the knurled nut no. 0230/15 and remove the caliper arm.
- Insert the other arm in the base and tighten the knurled nut. Please see to it that the pin at the base fits exactly into the groove of the caliper arm.
- Adjust the instrument :
 - Loosen the knurled nut no. 0230/29 at the upper measuring contact.
 - Turn the upper measuring contact no. 1530/01 so that both pointers indicate approximately the zero position. Tighten the knurled nut no. 0230/29.
 - Turn the scale until the big pointer indicates the zero position.
 - The small pointer does not have to indicate exactly the zero position.

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